

## To Assemble:

- Screw cap on to Sharps Container
- 

## To Dispose of Sharp when Sharps Dart is in Vertical Position:

- Using one hand, place the sharp, point first into the MedSource Sharps Dart until sharp is securely within the container, then release sharp.

**NOTE: DO NOT PLACE THUMB/ FINGERS INTO THE CONTAINER.**

- To close the Sharps Dart, grasp the container beneath the opening, making sure your hand and fingers are clear of opening.

- For closure, snap cap onto top, listening for snap.
- 

## To Dispose of Sharp when Sharps Dart is in Horizontal Position:

- Using one hand, place the sharp, point first, into the MedSource Sharps Dart until sharp is securely within the container, then release sharp.

**NOTE: DO NOT PLACE THUMB/ FINGERS INTO THE CONTAINER.**

- To close the container, grasp the container beneath the opening, making sure your hand and fingers are clear of opening, then lift the container to a vertical position. Make sure the sharps sit inside the container and do not project above the opening.

- For closure, snap cap onto top, listening for snap.
- 

## CAUTION:

The Sharps Dart should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container. User should carry sufficient number of containers to adequately accommodate disposal from anticipated usage. Sharps Container should not be placed in areas with unsupervised patient access.

## WARNING: Destroy by incineration.