

ClearSafe Comfort® CS Safety IV Catheters

with **Slide Style** Safety Activation Technology

MEDSOURCE

procedure


Peel away label to remove the catheter.



Verify the push off tab and needle bevel are in the up position.
Do not rotate the catheter, as it may change the optimized heel distance.



MedSource catheters are intended to be used with an approach angle to insertion of 30 degrees or less.

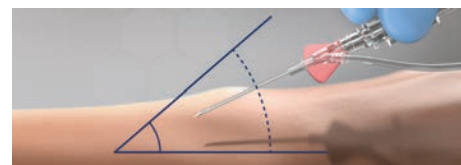


Insert needle into skin and vein and observe for blood flashback.




The inclination should be decreased to an angle appropriate to the depth and size of the intended vessel.



Holding the flash chamber, place finger on primary push off tab and thread desired catheter length into vein.

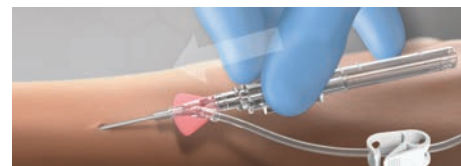

Keeping the device stable, retract needle housing until it “clicks” into place. This is the indication that the needle is now locked inside the safety guard.

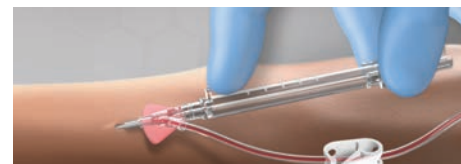

Once the needle is locked in place, the catheter hub will automatically disengage.



ClearSafe Comfort® BC Safety IV Catheters

with **Slide Style** Safety Activation Technology

MEDSOURCE

procedure

Following your facility's aseptic technique protocol,
connect syringe to the extension tube.



Set the pinch clamp.



Remove the syringe.



Use tape to secure IV port or follow your facility's standard
protocol for securement.

