

ClearSafe and ClearSafe Comfort® Safety IV Catheters

with **Slide Style** Safety Activation Technology

Peel away label to remove the catheter.

Verify the push off tab and needle bevel are in the up position.
Do not rotate the catheter, as it may change the optimized
heel distance.

MedSource catheters are intended to be used with an approach
angle to insertion of 30 degrees or less.

Insert needle into skin and vein and observe for blood flashback.

The inclination should be decreased to an angle appropriate
to the depth and size of the intended vessel.


Holding the flash chamber, place finger on primary push off
tab and thread desired catheter length into vein.

Keeping the device stable, retract needle housing until it “clicks”
into place. This is the indication that the needle is now locked
inside the safety guard.

Remove the needle guard by gently twisting and pulling it
out of the catheter hub. Secure the catheter.

MEDSOURCE

procedure

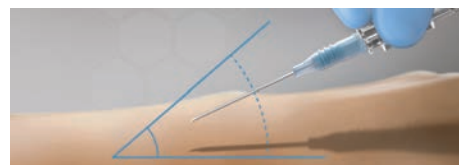

ClearSafe and ClearSafe Comfort® Safety IV Catheters

with **Slide Style** Safety Activation Technology

Connect the luer of the flushed IV administration line to the hub of the IV catheter.

Verify secure connections to prevent flow obstruction by twisting luer lock in place.

Use tape to secure IV port or follow your facility's standard protocol for securement.

MEDSOURCE

procedure

