


IV Catheter Dressing

1 Dressing/Pkg.

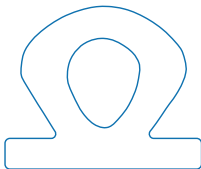

INSTRUCTIONS FOR USE

1. Clean and dry the wound area.
2. Remove dressing from wrapper and peel the paper backing from the adhesive.
3. Position the absorbant pad centrally over the affected area.
4. Press down the adhesive firmly and then remove the clear covering on the top.
5. Replace as required.

⚠ WARNING

Store in clean, dry conditions away from excessive heat, cold or light.

⚠ CAUTION

Federal Law (USA) restricts this device to sale by or on the order of a physician.

STERILE UNLESS PACKAGING IS OPEN OR DAMAGED

STERILE EO



SINGLE USE ONLY



MD

LOT



UDI



(01) 0 0816703 02451 5

PEEL BACK TO OPEN



Manufactured for:
MedSource Labs, LLC
8600 Shelby Court
Chanhassen MN, 55317 USA



MADE IN CHINA

MEDSOURCE

www.medsourcelabs.com

MSL-REG-LQR-8182, REV. 04