

Site-Loc

REF MS-11121

Transparent Dressing
2 3/8 in x 2 3/4 in
6 cm x 7 cm

1 Dressing/Pkg.

PEEL BACK TO OPEN



STERILE EO

 **SINGLE USE ONLY**

Not made with
natural rubber latex.

MD

Instructions:

1. Open package and remove sterile dressing.
2. Peel liner from dressing exposing adhesive surface
3. Slowly remove paper while continuing to smooth outer edges of dressing.
4. Smooth dressing from center to edges using firm pressure to enhance adhesion.
5. Secure any catheter, lumens, or extensions.

UDI



LOT:
EXP:
CONTROL:

(01) 0 0816703 02456 0

WARNING

Store in clean dry conditions away from excessive heat, cold or light.

CAUTION

Federal Law (USA) restricts this device to sale by or on the order of a physician.

STERILE UNLESS PACKAGING IS OPEN OR DAMAGED

Manufactured for
 MedSource
8600 Shelby Court,
Chanhassen, MN 55317 USA

MEDSOURCE 
www.medsourcelabs.com