

Instruction for Use

Face Shield

Model: Disposable **Size: 33*19 cm**

Regulation and standard: Comply with the European Parliament and Council Regulation(EU) 2016/425 on personal protective equipment(PPE) and be CE marked in accordance with the regulation. Comply with DS/EN ISO 16321-1:2022 Eye and face protection for professional use- Part 1: General requirements.

Range of application: The Disposable Face Shield is intended for use in situations where social distancing isn't feasible and in areas with high risk of community disease spread. The disposable face shield offers enhanced protection against the transmission of droplets in the air. This face shield provides an anti-fog property to reduce moisture build-up and avoid low visibility and hazing .Also protects the wearer from potentially contaminated fluid splash.

Structure and composition: The disposable face shield is made of clear plastic and be designed, produced, treated or finished so that it does not fog, is anti static and does not produce reflections that are s a nuisance to the user. The curved foam band and elastic band will keep the face shield stick to the face during use.Product is free from latex, PVC and nickel.

Storage requirements: Store in a clean, dry and well-ventilated environment in original packaging, away from contaminated areas, dust, sunlight and damaging chemicals.

Recycling: Packaging is sorted as paper. The product as general waste.

Caution: Do not use as protection against impacts from solid objects. Do not use a shield against substances capable of burning, corroding or dissolving by chemical action. Do not use as protection against a strong light source, gases or vapours of any kind, sparks, radiation or ultraviolet rays. Do not use in proximity to flames or hot objects.

Instructions: Without touching the clear polyester visor film, hold the face shield headband corner ends with both hands. Keeping hold of the corner end, place the face shield over the top of your forehead by pulling the elastic headband. Adjust the face shield to a comfortable position until the face shield feels secure. To remove the face

shield, lift the face shield over your head and drop it in the appropriate waste receptacle. Do not touch the visor at any point during the removal process. Wash or disinfect hands after use.