

Paper Tape REF: MS-15410 [Information For Use]

Application of Tape:

Prior to application check if the skin is dry and clean.

- 1. Apply tape smoothly.
 - Avoid creases or wrinkles.
- 2. Once tape has been applied, look to make sure surrounding skin's circulation is good.
 - Remove tape, if too tight.

Tape Removal:

Check for irratated or red skin.

- 1. Remove tape if any skin is irritated or red.
- 2. DO NOT pull or rip tape off aggressively.
- 3. Tape can be removed easily with water.



$ar{!}\setminus$ caution:

- 1. Avoid irritated, infected, or open wounds near skin to be taped.
- 2. Avoid taping skin if the patient has any history of hyper-sensitive skin or any pre-existing skin conditions.
- 3. Avoid any sensitive skin areas, or compromised circulation.

