

## CPR Barrier Shield



**SINGLE USE ONLY**

### Intended Use:

This CPR BARRIER SHIELD is intended to provide a physical barrier between rescuer and patient during the administration of CPR on adults and children weighing more than 22 lbs.

### Instructions For Use:

1. Insert the airway tunnel into the patient's mouth.
2. Placing your fingers beneath the shield, pinch patient's nasal airway closed.
3. Using the recommended head-tilt/chin-lift method, establish an open airway.
4. Form an airtight seal between the barrier shield and the patient's mouth.
5. Begin CPR ventilation using currently accepted technique.



**CAUTION:** For use only by persons trained in currently accepted CPR techniques.



**WARNING:** When used properly, this product provides a physical barrier during mouth-to-mouth resuscitation. It does not guarantee absolute protection from disease transmission.



Manufactured for:

MedSource  
8600 Shelby Court  
Chanhassen, MN 55317

LOT:

EXP:

CONTROL:



(01) 0 0816703 02582 6



Made in China REV. 04