

# MEDSOURCE

## Nebulizer

Large Volume

With Tee, Tubing and Mouthpiece

**MS-22893**


Not made with natural rubber latex.



SINGLE USE ONLY

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.

**⚠ CAUTION:** Federal law (USA) restricts this device for sale by or on the order of a physician.

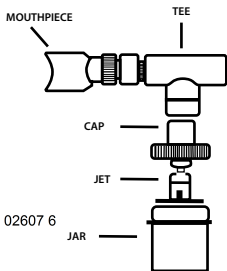

(01) 0 0816703 02607 6

LOT




REV. 03

CORRUGATED TUBING



SUPPLY TUBING

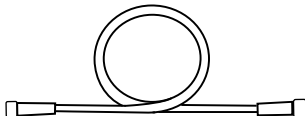

Made in China

## Directions for Use:

**NOTE:** These directions are general guidelines intended for use by qualified medical personnel.



**WARNING:** Be sure all connections are secure.

1. Place prescribed solution into the nebulizer jar.
2. Attach cap to nebulizer jar with a clockwise rotation.
3. Insert the nebulizer into tee.
4. Connect the mouthpiece to the tee. Discard mouthpiece if not used.
5. Attach the supply tubing between the gas source and nebulizer.
6. Set gas flow to 4-8 LPM. Check for aerosol mist; if necessary, tap the device until it begins to nebulize.
7. During treatment, periodically tap nebulizer to minimize residual volume.



**Manufactured for:**

**MedSource**

**8600 Shelby Court**

**Chanhassen, MN 55317 USA**

**[www.medsourcelabs.com](http://www.medsourcelabs.com)**



**WARNING:** This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm.

For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).