




Adult

Not made with
natural rubber latex.
SINGLE USE ONLY

**Nebulizer
Elongated Aerosol, Mask and Tubing
MS-22895**

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.

**⚠ CAUTION: Federal law (USA) restricts this device for sale by
or on the order of a physician.**

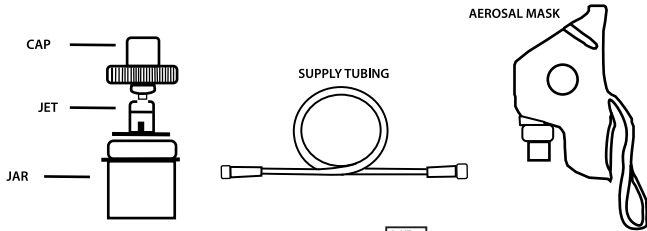

UDI



(01) 0 0816703 02609 0

LOT



Directions for use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.



WARNING: Be sure all connections are secure.

1. Place prescribed solution into the nebulizer jar.
2. Attach cap to nebulizer jar with a clockwise rotation.
3. Attach the supply tubing between the gas source and the nebulizer.
4. Connect the nebulizer to the mask inlet.
5. Set gas flow to 4-8 LPM. Check for aerosol mist; if necessary, tap the device until it begins to nebulize.
6. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
7. Gently pull the ends of the strap until the mask is secure. Mold the metal strip on the mask to fit the face.
8. During treatment, periodically tap nebulizer to minimize residual volume.



Manufactured for:
MedSource Labs, LLC
8600 Shelby Court
Chanhassen MN, 55317 USA
Made in China



WARNING: Cancer and Reproductive Harm.
For more information visit www.P65Warnings.ca.gov.