

TEAR AT NOTCH

MEDSOURCE

MS-23950

Robertazzi Nasopharyngeal Airway

SIZE: 20FR

-
- Sterile
 - Not Made With Natural Rubber Latex
-

DIRECTIONS:


1. Lubricate the airway with water soluble lubricant.
 2. Gently insert airway into nostril.
 3. Do not force. Gently rotate the airway to ease entry.
-

⚠ CAUTION:

- Federal (USA) law restricts this device to sale by or on order of a physician.
- Replace per CDC guidelines and/or hospital policy.
- For single use only. Do not resterilize.



SINGLE USE ONLY

 MedSource
www.medsourcelabs.com
 Made in Malaysia



(01) 0 0816703 02654 0

LOT

