Tear at notch **MED**SOURCE: MS-23970 Silicone Nasopharyngeal Airway SIZE: 20FR Sterile Not made with natural rubber latex Directions: $1. \, Lubricate \, the \, airway \, with \, water \, soluble \, lubricant.$ 2. Gently insert airway into nostril. 3. Do not force. Gently rotate the airway to ease entry. ⚠ CAUTION: • Federal (USA) law restricts this device to sale by or on order of a physician. • Replace per CDC guidelines and/or hospital policy. • For single use only. Do not resterilize. LOT Manufactured for: 8600 Shelby Court Chanhassen MN, 55317 USA MedSource www.medsourcelabs.com ☑ Made in China **REV. 01**