

150mm



MEDSOURCE 

Nasal Cannula with Flared Tips

7' Tubing

ADULT

MS-24004

Qty:1

LOT



UDI



(01) 0 0816703 02687 8



CAUTION: Federal Law (USA) restricts this device for sale by or on the order of a physician.

Directions for Use:

1. Attach the oxygen supply tubing to the gas source and set the oxygen to prescribed flow.
2. Check for gas flow through the device.
3. Insert the nasal tips into the nostrils with the two plastic tubes over the ears and under the chin.
4. Gently adjust the plastic slide until the cannula is secure.

NOTE: Nasal tips may be trimmed with scissors to fit smaller patients.

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.



WARNING: Cancer and Reproductive harm. www.P65Warnings.ca.gov.



SINGLE USE ONLY

**Not made with
natural rubber latex.**



Manufactured for:
MedSource Labs, LLC
8600 Shelby Court
Chanhassen, MN 55317
www.medsourcelabs.com
Made in China

MD

MSL-REG-LQR-7072
REV.03

220mm