

150mm



**MEDSOURCE** 

Nasal Cannula with 14' Tubing

**ADULT**

**MS-24005**

Qty: 1

LOT

UDI



(01) 0 0816703 02689 2



CAUTION: Federal Law (USA) restricts this device for sale by or on the order of a physician.

**Directions for Use:**

1. Attach the oxygen supply tubing to the gas source and set the oxygen to prescribed flow.
2. Check for gas flow through the device.
3. Insert the nasal tips into the nostrils with the two plastic tubes over the ears and under the chin.
4. Gently adjust the plastic slide until the cannula is secure.

NOTE: Nasal tips may be trimmed with scissors to fit smaller patients.

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.



WARNING: Cancer and Reproductive harm. [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).



SINGLE USE ONLY

Not made with  
natural rubber latex.  
Not made with DEHP.



Manufactured for:  
MedSource  
8600 Shelby Court  
Chanhassen, MN 55317  
[www.medsourcelabs.com](http://www.medsourcelabs.com)  
Made in China

MD

MSL-REG-LQR-7070  
REV.04

220mm