

MEDSOURCE



Nasal Cannula

7' Tubing w/ Universal Connector

MS-24008



Adult

Not made with
natural rubber latex.

MD

SINGLE USE ONLY

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.



CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

Directions for Use:

1. Attach the oxygen supply tubing to the gas source and set the oxygen to prescribed flow.
2. Check for gas flow through the device.
3. Insert the nasal tips into the nostrils with the two plastic tubes over the ears and under the chin.
4. Gently adjust the plastic slide until the cannula is secure.

NOTE: Nasal tips may be trimmed with scissors to fit smaller patients.



WARNING: Cancer and Reproductive Harm. www.P65Warnings.ca.gov.

LOT

UDI



(01) 0 0816703 02693 9

