

MS-24212-U

Nasal Cannula

with soft-tips

Universal Connector



(01) 0 0816703 02998 5

Lot Number:

Control:

Expires:



DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.

CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

Directions for Use:

Attach the oxygen supply tubing to the gas source and set the oxygen to the prescribed flow. Check for gas flow through the device. Insert the nasal tips into the nostrils with the two plastic tubes over the ears and under the chin. Gently adjust the plastic slide until the cannula is secure.

NOTE: Nasal tips may be trimmed with scissors to fit smaller patients.



Manufactured for:

MedSource

8600 Shelby Court

Chanhassen MN, 55317 USA

Not made with natural rubber latex.

Made in China



WARNING: This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm.

For more information go to www.P65Warnings.ca.gov.

