

# MEDSOURCE



## Infant

Not made with  
natural rubber latex.  
SINGLE USE ONLY




## Non-Rebreathing Mask

Universal Connector

### MS-25055-U

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.

**⚠ CAUTION:** Federal law (USA) restricts this device for sale by or on the order of a physician.

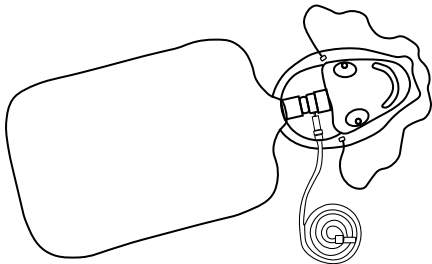

LOT




(01) 0 0840155 40002 3



REV. 02

## Directions for Use:

**NOTE:** These directions are general guidelines intended for use by qualified medical personnel.

 **WARNING:** Always check oxygen flow before placing the mask on the patient. The mask has no provision for entrainment of room air should the oxygen flow cease. Check that all valves are functioning and that oxygen inlet valve moves freely.

1. Attach the oxygen supply tubing to the gas source.
2. Set the oxygen to the desired flow.
3. Check for gas flow through the device. Always verify proper function of the valves. (The valves between the mask and the reservoir should rise on inspiration and lower on exhalation.)
4. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secure.
5. Mold the metal strip on the mask to fit the face.



**Manufactured for:**  
**MedSource**  
**8600 Shelby Court**  
**Chanhassen, MN 55317 USA**  
**[www.medsourcelabs.com](http://www.medsourcelabs.com)**



**Made in China**



**WARNING:** This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm.  
For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).