

MEDSOURCE



Non-Rebreathing Mask

Partial Non-Rebreathing Mask

MS-25059



Adult


Not made with natural
rubber latex.

Not made with DEHP.
SINGLE USE ONLY

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING



CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

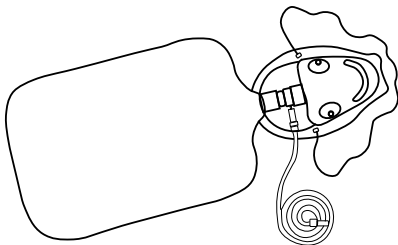

(01) 0 0840155 40014 6

LOT



REV. 01

Directions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

⚠ WARNING: Always check oxygen flow before placing the mask on the patient. The mask has no provision for entrainment of room air should the oxygen flow cease. Check that all valves are functioning and that oxygen inlet valve moves freely.

1. Attach the oxygen supply tubing to the gas source.
2. Set the oxygen to the desired flow.
3. Check for gas flow through the device. Always verify proper function of the valves. (The valves between the mask and the reservoir should rise on inspiration and lower on exhalation.)
4. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secure.
5. Mold the metal strip on the mask to fit the face.



Manufactured for:

MedSource Labs

8600 Shelby Court

Chanhassen, MN 55317 USA

www.medsourcelabs.com



Made in China