

MEDSOURCE



Oxygen Mask

High Concentration

MS-25073



Pediatric

Not made with
natural rubber latex.

SINGLE USE ONLY

MD

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING

CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.


WARNING: Cancer and Reproductive harm.
www.P65Warnings.ca.gov.

UDI



(01) 0 0840155 40022 1

LOT

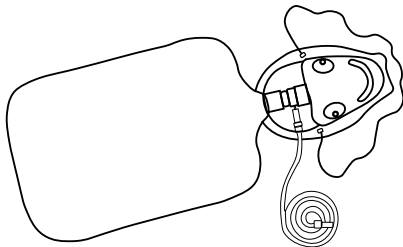

Directions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.



WARNING: Always check oxygen flow before placing the mask on the patient. The mask has no provision for entrainment of room air should the oxygen flow cease. Check that all valves are functioning and that oxygen inlet valve moves freely.

1. Attach the oxygen supply tubing to the gas source.
2. Set the oxygen to the desired flow.
3. Check for gas flow through the device. Always verify proper function of the valves. (The valves between the mask and the reservoir should rise on inspiration and lower on exhalation.)
4. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secure.
5. Mold the metal strip on the mask to fit the face.



Manufactured for:
MedSource Labs, LLC
8600 Shelby Court
Chanhassen MN, 55317 USA
www.medsourcelabs.com



Made in China