

MEDSOURCE



Aerosol Mask MS-25080



Adult

Not made with
natural rubber latex.
Not made with DEHP.
SINGLE USE ONLY

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING

⚠ CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

Directions for Use:

1. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
2. Gently pull the ends of the strap until the mask is secure.
3. Mold the metal strip on the mask to fit the face. For small volume nebulizers, attach nebulizer to mask; for large volume nebulizers, attach one end of corrugated tube to the mask and other end to nebulizer.

LOT



REV. 01



Manufactured for:
MedSource
8600 Shelby Court
Chanhassen, MN 55317 USA
www.medsourcelabs.com
 Made in China



(01) 0 0840155 40024 5