

# MEDSOURCE

**Aerosol Mask  
Elongated  
MS-25083**

**Pediatric**



Not made with  
natural rubber latex  
Not made with DEHP  
SINGLE USE ONLY

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING



**CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.**

### **Directions for Use:**

1. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
2. Gently pull the ends of the strap until the mask is secure.
3. Mold the metal strip on the mask to fit the face. For small volume nebulizers, attach nebulizer to mask; for large volume nebulizers, attach one end of corrugated tube to the mask and other end to nebulizer.

LOT



REV. 01



(01) 0 0840155 40026 9

**Manufactured for:**  
**MedSource**

**8600 Shelby Court  
Chanhassen, MN 55317 USA  
[www.medsourcelabs.com](http://www.medsourcelabs.com)**

Made in China

