

MEDSOURCE



Adult

Not made with
natural rubber latex.
SINGLE USE ONLY

Tracheostomy Mask MS-25100



DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.



CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

NOTE: For small volume nebulizers, attach the nebulizer to mask; for large volume nebulizers, attach one end of corrugated tube to the mask and other end to nebulizer.

Directions for Use:

NOTE: Unsnap the strap before use.

1. Place the mask on the patient's tracheostomized area with the elastic strap around neck.
2. Then snap the strap.
3. Gently pull the ends of the strap until the mask is secure.



WARNING: This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov.

LOT



REV. 03



(01) 0 0840155 40030 6

Manufactured for:

MedSource Labs

8600 Shelby Court

Chanhassen, MN 55317 USA

www.medsourcelabs.com

Made in China

