



www.medsourcelabs.com

ADULT

Not made with
natural rubber latex.

 SINGLE USE ONLY

Cat. No. MS-25210-U
Tracheostomy Mask Kit
All Diluters, Universal Connector


DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.

CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

Lot Number:

Control:

Expires:

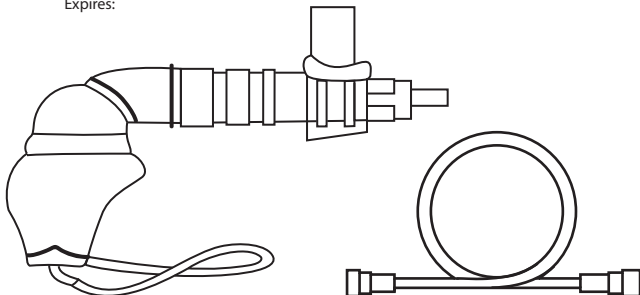

Directions for Use:

- Insert selected diluter into the corrugated tubing.
- Connect adaptor to the diluter.
- Attach the supplying tubing between the gas source and the diluter.
- Unsnap the strap, place the mask on the patient's tracheostomized area with the elastic strap around neck, then snap the strap. Gently pull the ends of the strap until the mask is secure.
- Set gas flow as follows:

Color	Blue	White	Orange	Yellow	Red	Pink
Setting	24%	28%	31%	35%	40%	50%
Suggested Oxygen Flow	4LPM	4LPM	6LPM	8LPM	8LPM	10LPM



MedSource
8600 Shelby Court
Chanhassen, MN 55317 USA
Made in China



WARNING: This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm.
For more information go to www.P65Warnings.ca.gov.