

MEDSOURCE

Elongated Oxygen Mask Medium Concentration MS-26041

Adult

Not made with
natural rubber latex.



SINGLE USE ONLY

MD

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING



CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

Directions for Use:

1. Attach the oxygen supply tubing to the gas source and set the oxygen to prescribed flow.
2. Check for gas flow through the device.
3. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
4. Gently pull the ends of the strap until the mask is secure.
5. Mold the metal strip on the mask to fit the face.



Manufactured for:
MedSource Labs, LLC
8600 Shelby Court
Chanhassen MN, 55317 USA
www.medsourcelabs.com
Made in India

LOT



UDI



(01) 0 0840155 41205 7