

MEDSOURCE

Elongated Oxygen Mask Medium Concentration MS-26042

Pediatric

Not made with
natural rubber latex
SINGLE USE ONLY



MD

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.



CAUTION: Federal law (USA) restricts this device for sale by or on the order of a physician.

Directions for Use:

1. Attach the oxygen supply tubing to the gas source and set the oxygen to the prescribed flow.
2. Check for gas flow through the device.
3. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
4. Gently pull the ends of the strap until the mask is secure.
5. Mold the metal strip on the mask to fit the face.



WARNING: Cancer and Reproductive harm. www.P65Warnings.ca.gov.



MedSource
8600 Shelby Court
Chanhassen MN, 55317 USA
www.medsourcelabs.com

Made in China

LOT



UDI



(01) 0 0840155 40039 9

MSL-REG-LQR-7103, REV. 04