


Tear Here ----- Do not open until immediatley prior to use. ----- Tear Here

MEDSOURCE

MS-65004


Electrodes

Foam Electrodes
Conductive Adhesive Hydrogel

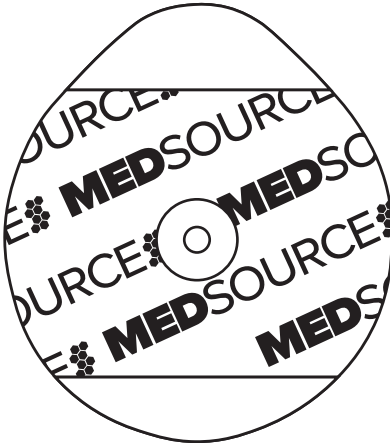

LOT



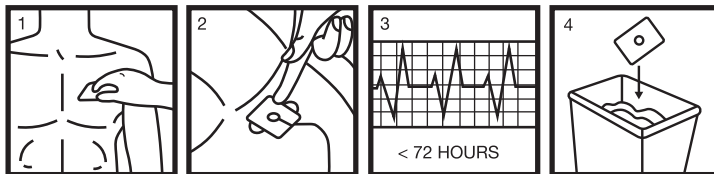

Prep
Preparare
Preparer
Preparera
Preparar

Apply
Faraderire
Adherir
Fast
Colar

Trace
Registrare
Trazar
Spara
Marcar

Dispose
Eliminare
Eliminar
Kassera
Eliminar

Attention:

The electrodes should be stored in non-corrosive gas, cool, and dry place.

Do not put the electrodes on mammilla, wrinkled skin, bony protuberances, wounds, or irritated skin.

If electrodes do not adhere properly to patient, apply a new pair.

Do not apply over broken or damaged skin.

The adhesive foam may be a mild irritant to the skin.

Follow required protocol.

Warning:

Do not use if conductive adhesive hydrogel is dried or damaged.

Misuse of electrodes may cause patient burns.



SEE INSTRUCTIONS



SINGLE USE



KEEP OUT OF
SUNLIGHT



KEEP DRY



MedSource Labs, LLC
8600 Shelby Court
Chanhassen MN, 55317 USA
www.medsourcelabs.com



Made in India