

Model: MS-90042 & MS-90042B



Disclaimer

This manual contains general instructions for the use, operation and care of this product. The instructions are not all-inclusive. Safe and proper use and care of this product are solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulation(s). It is recommended that training on the proper use of the MedSource Stair Chair be provided before using this product in actual situations.

Retain this manual for future reference. Include it with the product in the event of transfer to new users. Additional free copies are available upon request from Customer Service.

1.1 WARNING

- Improper use of the Stair Chair can cause injury. Use the Stair Chair only for the purpose described in this manual.
- Untrained operators can cause injury or be injured. Permit only trained personnel to operate the stair chair.
- An unattended patient can be injured. Stay with the patient at all times.
- An unrestrained patient can fall off the chair and be injured. Use restraints to secure the patient on the chair.
- Helpers can cause injury or be injured. Maintain control of the chair, operate the lock and direct any helpers.
- Improper maintenance can cause injury. Maintain the chair only as described in the user manual.
- Improper restraint attachment can cause the panels to dislodge, causing injury. Attach the restraints only as described in the User Manual.
- Improper use of the restraints can cause injury. Use the restraints only for the purpose described in the user manual.
- Improper parts can cause injury. Use only MedSource Stair Chair approved parts on the chair.
- Attaching improper items to the chair can cause injury. Use only MedSource approved items on the chair.

1.2 Important

Information in this manual provides or emphasizes important information and will be displayed as follows.

Water under high pressure can penetrate joints, flush away lubricant, and cause corrosion. Do not use a high-pressure washer or steam to clean the Stair Chair.

1.3 Blood Borne Disease Notice

OSHA (Occupational Safety and Health Administration) requires employers to protect workers from workplace exposure to blood borne diseases. To reduce the risk of exposure when using the Stair Chair follow the maintenance instructions in this manual.

For more information, contact www.osha.gov

2. About the Chair


2.1 Chair Description

The MedSource Stair Chair is an emergency patient-handling device designed to transport a patient in a seated position. The Stair Chair is for professional use by a minimum of two trained operators. It is recommended that additional help is to be used on stairs and with heavy patients.

The Stair Chair has telescoping carrying handles, vinyl seat, back cover, and one-piece restraints for the chest and thigh.

2.2 Chair Stair Specifications

Weight Rating.....	350 lbs	Overall Width.....	20 in (51 cm)
Seat Width:.....	17 in (43 cm)	Wheel Size.....	5 in (13 cm)
Seat Depth:.....	17 in (43 cm)	Strap Length.....	60 in (152 cm)
All Handles Extended.....	46 in (115 cm)	Folded Length.....	36 in (91 cm)
All Handles Retracted/ Folded...27 in (69 cm)		Folded Depth.....	7 in (18 cm)
Weight.....	23 lb (10.4 kg)	Maximum Load Limit.....	350 lb (159 kg)
Height.....	36 in (91 cm)		

 **IMPORTANT**
Inspect the chair if the load limit has been exceeded.

3. Using the Chair Features

3.1 Chair Lock

The chair lock automatically engages as the Stair Chair is completely unfolded. It secures the chair in an open position.

3.2 Unfolding the Stair Chair

1. Stand at the side of the Stair Chair.
2. Position the Stair Chair on its wheels.
3. Grasp the frame at the seat and back.
4. Briskly push the seat and back of Stair Chair.
5. Continue to push apart until Stair Chair locks into place.
6. Confirm locking by visually checking that both sides of the lock bar are engaged.

3.3 Folding the Stair Chair

1. Fold rear carrying handles down.
2. Retract from carrying handles.
3. Stand at one side of the Stair Chair.
4. Tilt Stair Chair forward onto front carrying handles.
5. Pull upward on center of lock bar to disengage lock.
6. Push frame down until Stair Chair is completely folded.

3.4 Rear Carrying Handles

To unfold the rear carrying handles for patient transport:
Grasp and pull upward on the handles.

To fold the handles down for storage:
Push the handle downward, towards the frame of the Stair Chair.

3.5 Front Carrying Handles

To extend front carrying handles:
Push button and pull handle outward.

To retract front carrying handles:
Push button and push into frame of the Stair Chair.

4. Operator Skill and Training

4.1 Skills

Operator using the chair needs:

- A working knowledge of the Stair Chair and emergency patient-handling procedures.
- The ability to assist the patient.
- A complete understanding of the procedures described in the Stair Chair User Manual.

4.2 Training

Trainees need to:

- Follow a training program designed by their training officer.
- Read the MedSource Stair Chair User Manual.
- Practice with the Stair Chair before using it in regular service.
- Be tested on their knowledge of the Stair Chair.

5. Using the MedSource Stair Chair

5.1 Before Placing the Stair Chair in service

Before placing your MedSource Stair Chair into use,
please confirm that the chair opens and operates properly.

5.2 General Guidelines for Use:

1. The Stair Chair is for professional use only.
2. Using the Stair Chair requires a minimum of two trained operators.
3. Operators will need help on stairs when working with heavy patients.
4. Follow standard emergency patient-handling procedures when operating Stair Chairs.
5. Always use patient restraints
6. Restrain the patient with MedSource International approved restraints.



IMPORTANT

DO NOT restrain the patient with alternate materials such as gauze, tape or elastic bandages that may interfere with chair operation. Should the patient have an injury that makes using a MedSource International Stair Chair impractical, use a MedSource International Scoop Stretcher, backboard, or cot in-lieu of the Stair Chair.

5.3 Transferring the Patient to the Stair Chair

Make sure the Stair Chair lock is engaged before transferring the patient and always secure the patient in the chair with restraints. (Two restraints included)
Use local approved protocol to transfer the patient to the Stair Chair

5.4 Rolling the Chair

When appropriate, the MedSource Stair Chair can be rolled. To roll the MedSource Stair Chair:

1. Secure patient on chair with restraints.
2. Front Operator: Stand by to assist rear operator as needed.
3. Rear Operator: Firmly grasp the rear carrying handles and tilt chair back until the patient's weight is balanced on the rear wheels, then wheel the chair.
4. Both Operators: When crossing rough terrain or obstacles, grasp carrying handles to lift and carry Stair Chair.

5.5 Carrying the Stair Chair on Stairs

Follow these general guidelines when carrying the chair on the stairs:

- Use a third (assisting) operator when ascending and descending stairs.
- If one of the two operators is considerably taller than the other, position the shorter of the two at the rear of the chair and taller at the front.
- When carrying the stair chair both operators should face the patient.

To carry the MedSource Stair Chair:

1. Secure patient on the chair with restraints.
2. Front and Rear Operators: Move to positions at the front and rear of the chair.
3. Assisting Operators: When going down stairs, stand behind the front operators. When going up stairs, stand behind the rear operator.
4. Front Operator: (Always face the patient) Grasp and extend carrying handles, position the handles to fit comfortably.
5. Rear Operator: Grasp the rear carrying handles. Tilt chair back slowly, until weight is balanced on transport wheels.
6. Front and Rear Operators: Position Chair near first step. In a coordinated movement, lift chair and begin to carry it up or down the stairs.
7. Assisting Operators: Steady operator by placing your hand on their back. Provide verbal cues to help guide the operator up and down the stairs.

6. Maintaining the Stair Chair

6.1 Maintenance Schedule

The MedSource Stair Chair requires regular maintenance.

It is advised that the owner/ users set up and follow a maintenance schedule and keep maintenance records.

When using products for maintenance on the MedSource Stair Chair,
Follow the manufacturers directions and read the MSDS.

The Following table represents the recommended minimum maintenance:

	Each Use	As-Needed	Each Month
Disinfecting	*		
Cleaning		*	*
Inspecting		*	*
Lubricating		*	

6.2 Disinfecting

To disinfect the MedSource International Stair Chair's metal frame and vinyl cover, wipe or spray all surfaces with disinfectant. Follow the disinfectant manufactures directions for use. It is recommended that the Stair Chair is also inspected for obvious damage as it is being disinfected. MedSource recommends that a mild soap solution be used to clean the Stair Chair.



Phenol and iodine based disinfectants as well as bleach can harm the Stair Chair. Disinfect and clean the chair with product that does not contain these items.

Water under pressure can penetrate joints, flush away lubricant, and cause corrosion. It is recommended that a high pressure washer or steam is not used to clean the MedSource Stair Chair.

6.3 Inspecting the Stair Chair

MedSource recommends the following check list be completed when inspecting the Stair Chair.

- Are all components present?
- Are all screws, nuts, bolts and roll pins present and securely in place?
- Does the chair lock into place properly?
- Does the chair unlock easily?
- Does the chair roll smoothly?
- Do all the moving parts operate smoothly and properly?
- Is the vinyl cover in good condition with no cuts or frayed edges?
- Are restraint buckles free of visible damage?
- Do restraint buckles operate smoothly and properly?

It is recommended that if the inspection shows damage or excessive wear that the MedSource Stair Chair is removed from service until repairs can be made.

7. Warranty

7.1 Limited Warranty Summary

MedSource warrants the product we manufacture to be free from defects in material and workmanship for 1 year except follows:

- A. External finishes are warranted for 90 days
- B. Soft goods (webbing, vinyl, etc.) are warranted for 90 days

This limited warranty applies when you use and care for the product properly. If the product is not used and cared for properly, the warranty is void. The warranty period begins the day the product is shipped from MedSource or the day you receive it, if you have proof of the delivery date. Shipping charges are not covered by the limited warranty. We are not liable for shipping damages or damages sustained through misusing the product.