



www.medsourcelabs.com

Total Lifter  
MS-95200

## USER MANUAL

The MedSource Total Lifter is a transport unit rated to transport up to 450kg. The unit must be placed so the handle support straps are to the side opposite the patient. The Total Lifter is for use by professional staff for lifting and transporting/transferring patients. It is recommended to use at least four persons to provide safe transport/transfer, thus reducing the possibility of accident or injury. Grab handles in line with the people lifting on opposite sides of the unit (DO NOT CROSS HANDLES). The Total Lifter is specifically for use when there is no need for spinal stabilization or traction of the patient being transported. If the Total Lifter is damaged, frayed, cut, or soiled, replace with new unit.



### DIRECTIONS

The MedSource Total Lifter is designed as an emergency manual transfer sheet to be used by a minimum of four people, with six people being recommended. The Total Lifter lifts, moves, and extracts victims and patients.

Step 1: The sheet is positioned full length under the patient.

Step 2: Each person takes hold of two (one per hand) handles.

Step 3: Using only recommended/approved and proper safety technique, the patient is lifted and moved accordingly.

### Size & Weight

107 x 183cm; 900g

### Material

Polyester Fibers - non-woven and woven with antimicrobial properties

### Weight Rating

Supports up to 450kg

Single handle load rating is 110kg

### Fluid Resistance

Resistant to the following fluids:

Oils, alcohol, mild cleaning fluids, transmission fluids, and brake fluids



Made in China

### Packaging

1 per polybag

10 per carton

**Shelf Life:** Five years



MedSource Labs, LLC  
8600 Shelby Court,  
Chanhassen MN, 55317 USA  
952-472-0131  
www.medsourcelabs.com



Consult Instructions for Use



Caution, consult accompanying documents



Catalogue number



Do not reuse



Importer



Country of manufacture