



Place the MedSource Patient Mover Rescue Chair so that the straps are to the outside of the patient. The Patient Mover requires a minimum of 2 - 4 trained professionals for safely lifting and transferring patients. Handles should be held in line with the person on the opposite side of the unit. Do not cross handles.

This unit should not be used when a spinal injury is suspected.

**Be sure to read and follow all warning information.
Improper use can lead to serious injury or death.**

Warning

Do not cross handles, always grasp handles directly across from other user.

Discontinue use if punctured, torn or excessively worn.

Avoid contact with sharp objects while patient mover is in use.

Only use handles for lifting.

Use care to ensure grip is secure prior to lifting.

Do not store or come in contact with heat sources greater than 200° F.

Avoid dragging over rough surfaces.

Do not attach to mechanical lifts.

Minimum of 2-4 operators are required.

Patient Mover may be wiped down with disinfectant, detergent, soap or with a damp cloth. Avoid machine washing or drying.

Maximum capacity is 350 lbs (*Third Party Tested*).