

**REF** Model: MS-95220



### Description/ Intended Use

MedSource Fast Stretchers are specially designed for moving patients. It is great for transporting patients out of confined areas or stairways. The weight capacity of the Fast Stretcher is 226 kg. The Fast Stretcher is intended for limited use. If the unit is damaged, frayed or cut, replace with new unit. Do not machine wash or dry.



### Instructions

Place Fast Stretcher under patient. Lift patient gently with handles. A minimum of 2 operators are needed to lift the patient, 4 - 6 operators are recommended. Not intended for use with suspected spinal injury patients.

Only qualified and properly trained personnel should use MedSource Fast Stretchers. Any persons using the Fast Stretchers should practice application and be thoroughly familiar with all medical protocols and aspects of its use before using. These instructions are provided as a guide only. Use protocols and techniques established by your local medical control.



Consult instructions for Use



Non sterile



Batch number



Caution, consult accompanying documents



Manufacturer



Catalogue number