

INSTANT COLD PACK

Instructions For Use:



1. Locate inner liquid pouch
Squeeze firmly to rupture



2. Shake to mix
contents



3. Apply to patient



4. Discard it

MS-CD6X8



SINGLE USE ONLY

INSTANT COLD PACK

CAUTION

- For external use only. Adult supervision recommended.
- Do not puncture. If bag breaks and fluid contacts skin, or eyes, immediately flush area with large amounts of water.
- Do not swallow contents. If contents accidentally swallowed drink large amounts of water (not milk) and contact Poison Control Centre or physician.
- Cold therapy should not be used by individuals with circulatory problems, unless under the direct supervision of a physician.



SINGLE USE ONLY